

Secuencia de Fresado

Fresas	
Ø 2.0	
Ø 2.5	
Ø 2.8	
Ø 3.2	
Ø 3.65	
Ø 4.2	
Ø 4.8	

HUESO TIPO I							
Ø							
3.5	100%	100%	80%	40%			
3.75	100%	100%	100%	80%	40%		
4.2	100%	100%	100%	100%	80%	40%	
5.0	100%	100%	100%	100%	100%	80%	40%

HUESO TIPO III							
Ø							
3.5	100%	100%	40%				
3.75	100%	100%	100%	40%			
4.2	100%	100%	100%	80%	40%		
5.0	100%	100%	100%	100%	80%	40%	

100%= Fresar aproximadamente el 100% del largo del implante a colocar
 80%= Fresar aproximadamente el 80% del largo del implante a colocar

HUESO TIPO II							
Ø							
3.5	100%	100%	40%	PC			
3.75	100%	100%	100%	40%	PC		
4.2	100%	100%	100%	80%	40%	PC	
5.0	100%	100%	100%	100%	80%	40%	PC

HUESO TIPO IV							
Ø							
3.5	100%	100%	PC				
3.75	100%	100%	100%	PC			
4.2	100%	100%	100%	40%	PC		
5.0	100%	100%	100%	100%	40%	PC	

40%= Fresar aproximadamente el 40% del largo del implante a colocar
 PC= Puntear la fresa sobre la cortical

Todas las dimensiones están en milímetros